



Molds IgG Panel

Please note: This test requires overnight shipping and the sample needs to be centrifuged. Please ring Invivo Clinical on 0333 241 2997 to advise us of the date of your Phlebotomy appointment so a FedEx collection can be arranged. Please book your blood draw Monday through to Wednesday only and in the morning. **Do not book on Thursdays and Fridays.**

- REDMOLP Assay
- Requested sample: Serum

Sample Preparation and shipping

- Collect blood in Tiger Top SST II Advance tube
- Allow to clot for 1hr
- Centrifuge (3000rpm 10 minutes)
- Aliquot the obtained serum in new clean transport tube. (3ml) Fill in label on the tube.
- **Ship immediately via overnight 24hr service. Does not require ice.**
- Place the vial in the biohazard bag. Seal the bag.
- Place the requisition form in the outside pocket of the biohazard bag - *please check that the collection date has been recorded on the TRF* (This is a regulatory requirement, ensuring that we receive a viable specimen and you receive accurate results).
- Place the biohazard bag into the box in which it came in.
- Place the box inside the Grey Returns envelope and into the FedEx Clinical Pak provided.
- Ensure sample is ready for collection by FedEx.