

Instructions

Urinary EcologiX<sup>™</sup>



Please read all instructions  
carefully before beginning

## HAZARDS & PRECAUTIONS

The sample collection tube contains boric acid, a preservative that ensures the urine sample is suitable for testing. **Please do not discard it.** Whilst the powder is non-hazardous, it can cause skin irritation and can be harmful if ingested, so please:


- + If you accidentally get powder on your hands, wash immediately with plenty of soap and water.
- + Do not ingest. If the powder (boric acid) is accidentally ingested, attend A&E immediately.
- + Keep out of the reach of children.

## IMPORTANT INFORMATION


- + Make sure your bladder is full before taking the sample. We recommend you collect the first urine of the day or, alternatively, do not drink excess fluid for 2 hours prior to collecting your sample.
- + Your sample needs to arrive at the laboratory between Monday to Friday and must be shipped within 24 hours of collection, ideally the same day it was collected. **Please do not refrigerate your sample before sending,** keep it at room temperature.
- + If you are also collecting a vaginal swab for the **Vaginal EcologiX™** test, please ensure that the vaginal swab is taken before the urine sample collection.
- + You must return your sample in the box that your test kit was sent in. This is a legal requirement and your sample may be rejected if this step is not followed.

## PATIENT PREPARATION

It is not recommended that you change any prescribed or regular medications or supplements without seeking advice from a suitably qualified medical professional. Some medications or supplements are known to have an impact on the test results. If agreed with your healthcare professional, please follow the guidelines below.

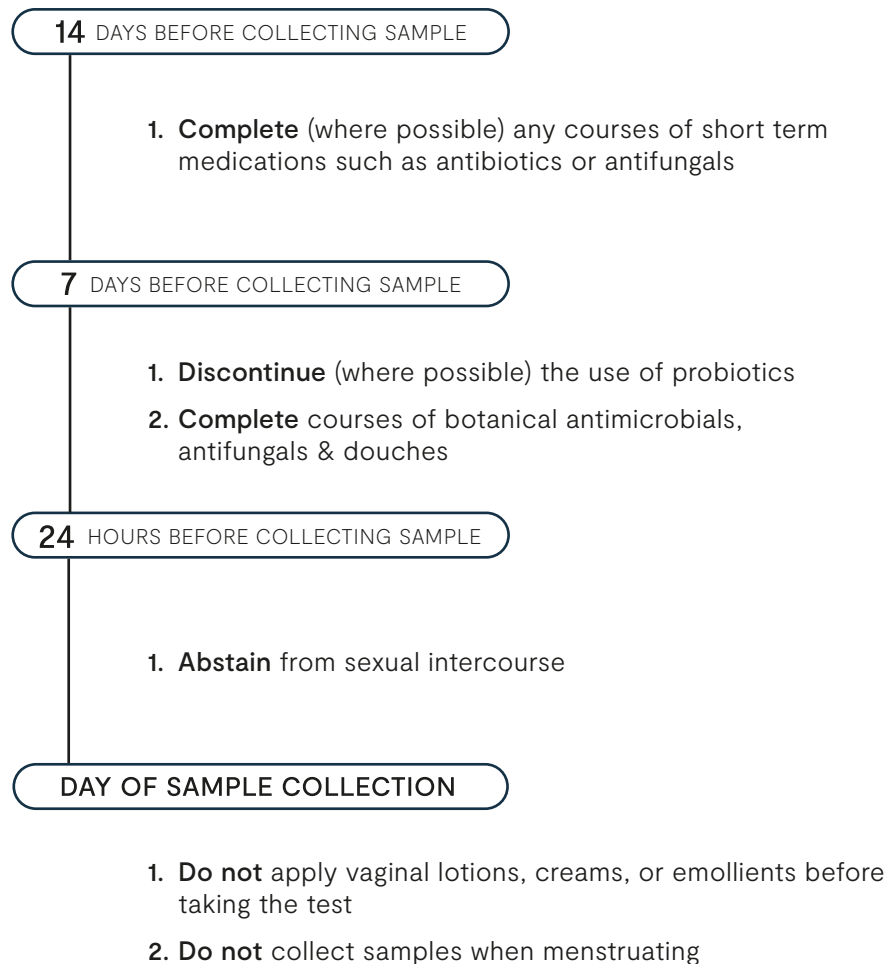

## KIT CONTENTS

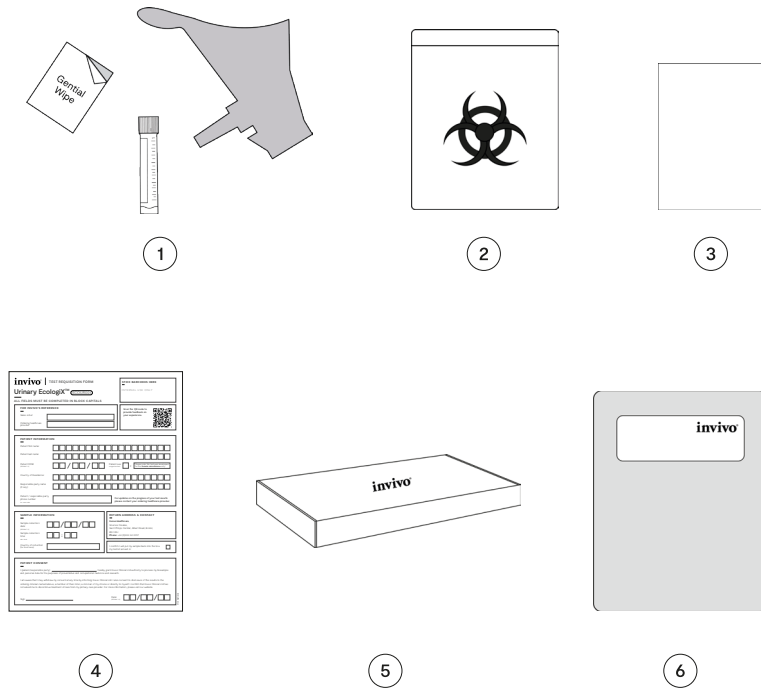

1. Peezy urine collection device
2. Biohazard bag
3. Absorbent white sheet
4. Test requisition form (TRF)
5. Test kit box (the box your test came in)
6. Unpaid mailing bag

## CHECKLIST

To ensure your samples are not rejected please ensure you have checked off every point in the below checklist.

CHECKLIST	
<input type="checkbox"/>	Read all instructions carefully before beginning
<input type="checkbox"/>	Complete test requisition form (TRF) <sup>4</sup>
<input type="checkbox"/>	Tightly secure the lid of the sample tube
<input type="checkbox"/>	Place the filled sample tube into the biohazard bag <sup>2</sup>
<input type="checkbox"/>	Place the completed TRF <sup>4</sup> into the side pocket of the biohazard bag <sup>2</sup>
<input type="checkbox"/>	Place the biohazard bag <sup>2</sup> into the test kit box <sup>6</sup> your kit arrived in
<input type="checkbox"/>	Put the test kit box into the mailing bag <sup>5</sup>
<input type="checkbox"/>	Arrange sample return




## HOW TO COLLECT YOUR SAMPLE


- Write in block capitals, the following information on the test requisition form <sup>④</sup>:  
**Your full name**  
**Your date of birth**  
**Date & time of collection**
- Wash your hands and remove the Peezy urine collection device <sup>①</sup> from its packaging.
- Write your **name, the date and time** of collection on the red topped bottle.
- Collect the urine sample following the instructions on page 7.
- Dispose of the Peezy urine collection device <sup>①</sup> in your household waste.
- Ensure the lid of your sample tube is tightly closed and place into the biohazard bag <sup>②</sup> and seal it closed.
- Place the completed test requisition form <sup>④</sup> into the side pocket of the biohazard bag <sup>②</sup>. Do not place it in the same pocket as the sample.
- The biohazard bag <sup>②</sup> should now include: the red topped tube containing your urine sample, the white absorbent sheet <sup>③</sup> and the completed test requisition form <sup>④</sup> in the side pocket.
- Place the biohazard bag <sup>②</sup> into the test kit box <sup>⑥</sup> it arrived in and place this into the addressed mailing bag <sup>⑤</sup>.**

## PEEZY URINE COLLECTION INSTRUCTIONS


- Wash your hands and remove the Peezy collection device <sup>①</sup> from its packaging.

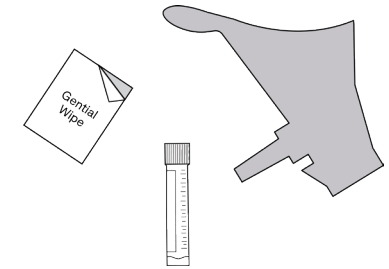

- Remove the lid from the red-topped container. **Do not discard the powder** and make sure to avoid contact with skin. Place the red cap on a clean surface ensuring the inside of the lid is facing upwards.

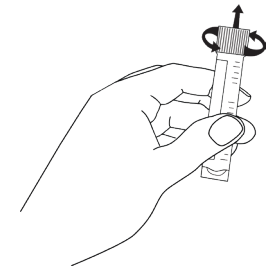

- Screw the tube onto the Peezy urine collection device securely as shown to right.

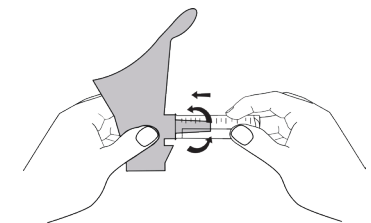


Instructions continue overleaf →




## PEEZY URINE COLLECTION INSTRUCTIONS


4. Use the genital wipe provided to clean the area around the opening of the urethra.

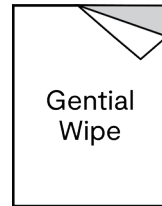

5. **Sit well back** on the toilet and hold the Peezy in place for urine collection, as shown.

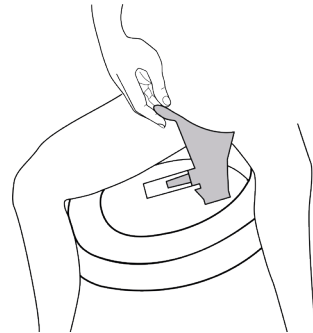


6. Start urinating. The first part of the urine will flow out of the bottom of the Peezy and then the tube will fill.

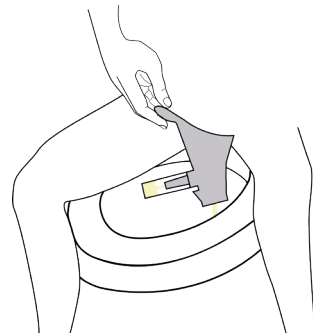


## PEEZY URINE COLLECTION INSTRUCTIONS

7. When you have finished, count to 10 – this lets the Peezy drain.

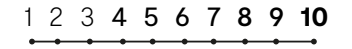


**Don't worry the collection device won't overflow**

8. Hold the tube containing the urine sample upright and detach the collection device. It is not necessary to pour any excess urine away – the “fill to” line is for guidance only.

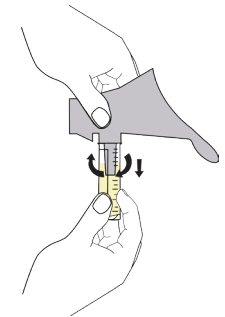

9. Throw away the Peezy device in your household waste

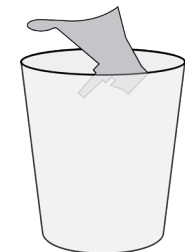


Instructions continue overleaf →




## PEEZY URINE COLLECTION INSTRUCTIONS

10. Screw the red cap back onto the collection tube.

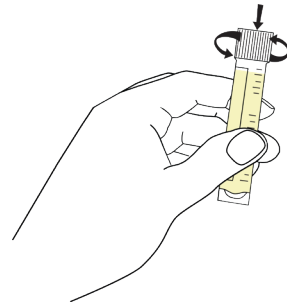

11. Mix the sample after collection by inverting the tube several times.

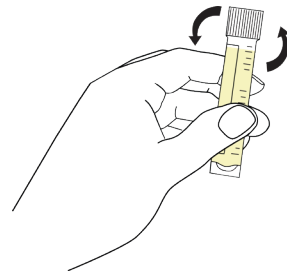

**Note: Please ensure the cap is tightly secured to prevent leaking!**

Please note the turnaround time for test results is 15 working days from the date the sample arrives at our lab.

Weekends & bank holidays are not included in the turnaround time.

For **video instructions** of how to use the Peezy urine collection device, scan this QR code:



**invivo**<sup>®</sup>

Invivo Healthcare, The Coach House, 3 Lewiston Mill, Brimscombe, GL5 2TE  
0333 241 2997 | [info@invivohealthcare.com](mailto:info@invivohealthcare.com)